

ERGOS RCA DUMMY

Dummy with hydraulic lifting system



Technical description


Dummy with a simple hydraulic lifting system that can be adapted to any boar individually. Reduces stress, injuries and time of the extraction process.

Benefits

- Time optimisation:**
 It facilitates the adaptation of the boar to the dummy, favoring better handling of the animal and saving time, as it adapts to the characteristics of any boar. This helps us to reduce mounting times.
- Fewer injuries:**
 The Ergos RCA also reduces injuries, so the number of boars with problems is decreased.
- Increased productivity:**
 With this system, both semen quality and ejaculate volume are improved. This is due to the optimal position of the boar, which implies a reduction in the efforts that the boar has to make during the process and consequently, a notable reduction in the stress level.
- Improved training and learning:**
 This system is ideal for the training of young boars. Thanks to the adaptability of the dummy, the boar can learn directly on the production dummy.
- With the same benefits of the Ergos dummy:**
 Robust dummy constructed of stainless steel for easy cleaning. Compatible with the Autocollect system. Possibility of transforming the Ergos dummy by adding the hydraulic system.

Installation example

Place the hydraulic unit on a vertical wall and anchor it with four screws.

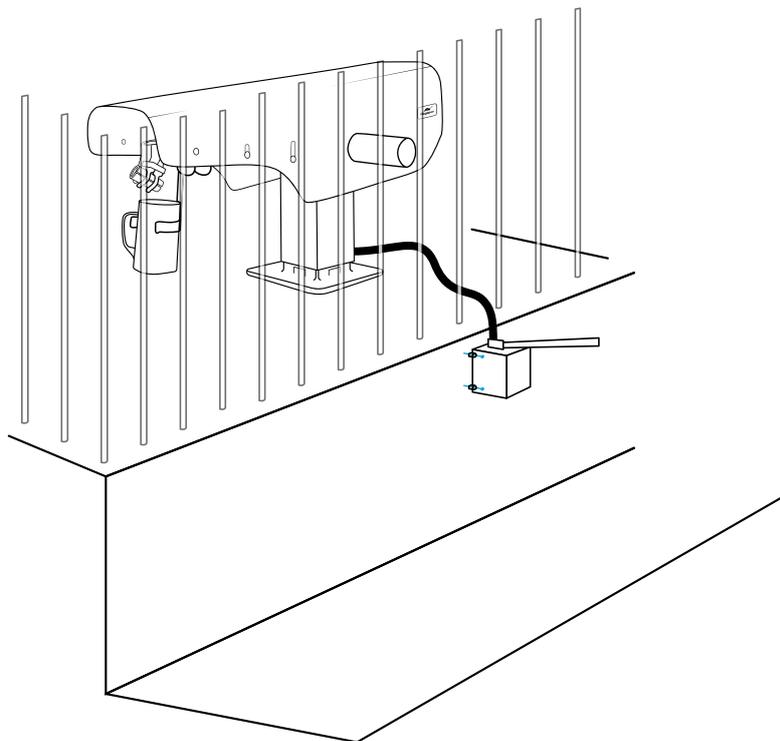

Use of the dummy:

1. Before the boar enters the extraction room, the dummy must be in the lowest position. To do this, the stopcock on the hydraulic unit must be opened.
2. Once the boar is on the dummy, close the stopcock to raise it.
3. Raise the dummy by moving the lever until the boar is suspended as much as possible. It is important to take into account the size of the boar to find the most comfortable position for him.
4. Once the ejaculation process has been completed, open the stopcock to allow the dummy to descend and facilitate the descent of the boar.