Magascan

Inalámbrico

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USER MANUAL

1. Introduction:

Magascan wireless scanners is the new generation instruments for veterinary ultrasonography with the outstanding feature of wireless and also interated with eartag number reading function.

Different with traditional veterinary ultrasound scanner with a cable connecting from probe to main unit, no cable appears at the end of the probe of Magascan. The probe of Magascan is highly integrated with ultrasound image processing eartag number reading, power management and a wireless signal provider to be connected by the main unit. The main units different with traditional devices are now changed to be any Android tablet device. The probe acted as a Wi-Fi Access Point and can be connected by Android device.

With the probe be connected through WIFI and the App is running, enjoy your days of working without the trouble making cables.

This manual is intended to provide a through overview of the Magascan Scanner and should be carefully read before starting operating the device.

Thank you for your trust in us to provide for your veterinary ultrasonography needs.

Image 1.1 Magascan Wireless ultrasound scanner

1.1 Features:

Image device: Android Smart Device

Scanning mode: 80° sector scanning

Probe: 3.5MHZTransducer

Scanning depth: 100-180mm

Display mode: B mode

Grey scale: 256 degree

Battery working time: >3 hours

Size: 193mm×47mm×35mm

Weight: 318g.

2. Before scanning:

Please read this manual carefully to avoid unnecessary risks.

2.1 Open the package

In order to prevent potential damage during shipping, Magascan wireless ultrasound probe has been tightly packed. Before opening the package, please make sure the package is in good condition and check according to below packing list to avoid anything missing.

Magascan packing list

Items	Included
Magascan wireless ultrasound probe	√
USB charger cable	√
wristband	√
Probe bag	√
charger	√

Each item should be examined for any noticeable defects or damage that may have occurred during shipment although it is packed carefully. If any defect or damage exists, please contact our local distributor/supplier to report the problem.

2.2 Install APP

If the wireless ultrasound probe APP is not installed, please contact with local distributor or supplier or install APP following these instructions:

4. Download the App software from website or USB flash disk.
5. Transfer the App software to your pad or phone.
6. Open your pad or phone, click "file"- " internal storage" then install the APP "UltraScan" software, as below pictures:

Image 2.2 Logo App Software

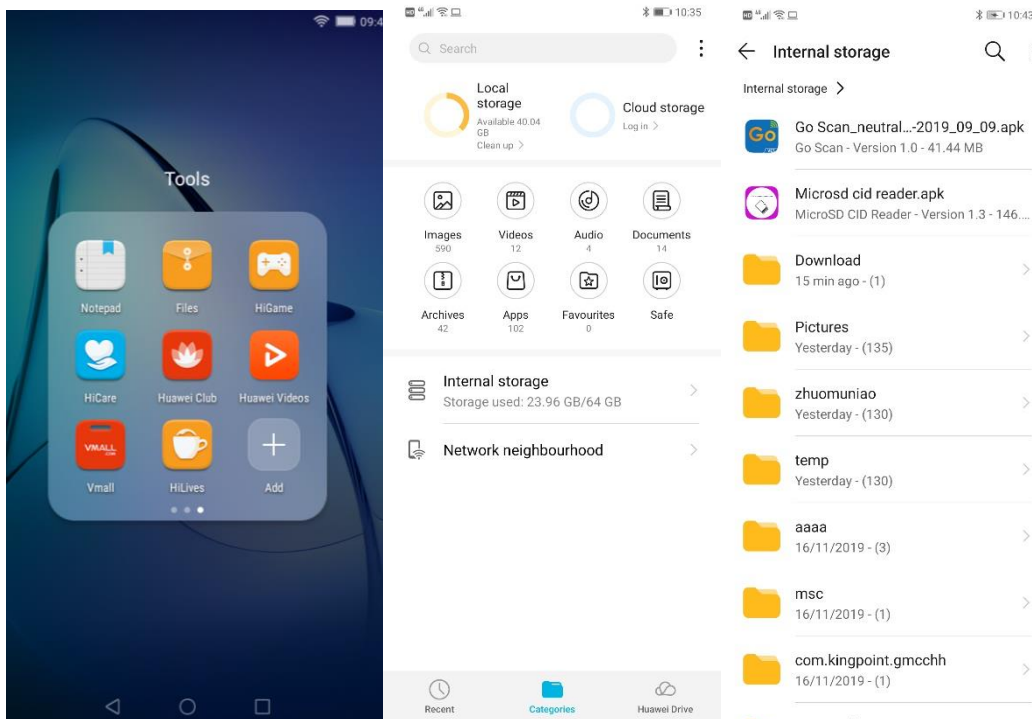
Image 2.2 Install App Software

Install the “UltraScan” software following the required steps, as below pictures.

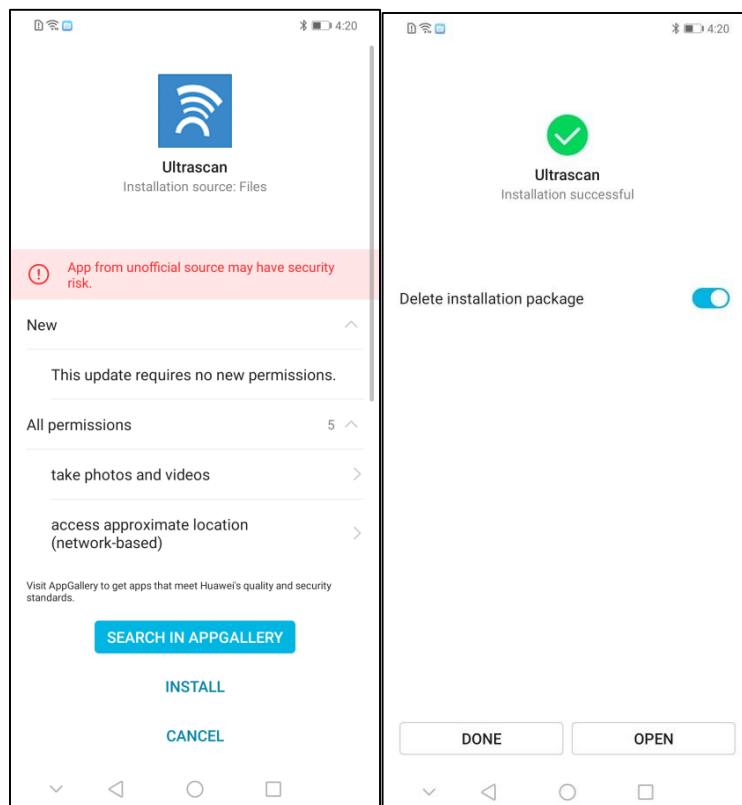
Image 2.2 Install App Software

Now the App software is ready for scanning

2.3 Connect to APP

1. Turn on the probe and connect to App

The wireless connection indication and battery circuit indication are not visible until the probe is turned on.

Click the button to turn on the probe. After the probe turns on, the battery power indicator will light up and indicate the current battery power. A few seconds after the probe is activated, the wireless indicator blinks, indicating that the wireless probe is ready to accept a connection from the display devices. The probe automatically detects and connects to the APP on the tablet or phone.

If there is no automatic connection after turning on the probe , it can be connected manually in the following steps:

1.1 Open the setting menu on tablet or phone.

1.2 Open "WLAN" setting menu

1.3 Click the WLAN of the probe, the WIFI names of different scanners are slightly different, similar to "CZTC1_2F109F6C".

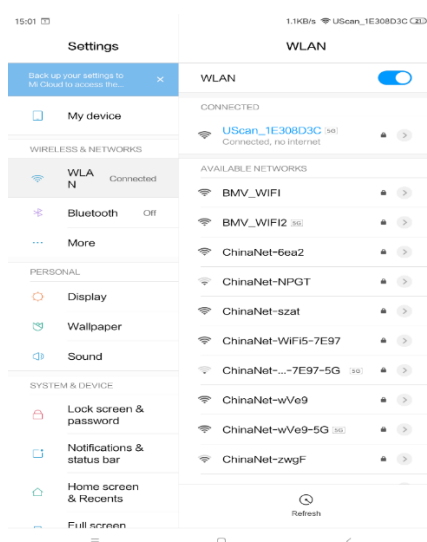
Image 2.3 App Software connection

2. Open the APP of tablet or mobile phone, and the probe has been connected successfully.

3. When you need to shut down the probe, please press the button, and keep a few seconds, after the probe is turned off, the battery indicator will go off.

Automatically detects and connects to the APP on the tablet or phone.

4. Enter the password to connect the probe. The password is same as probe ID.

3. Operate the APP:

3.1 Ultrasound scanning

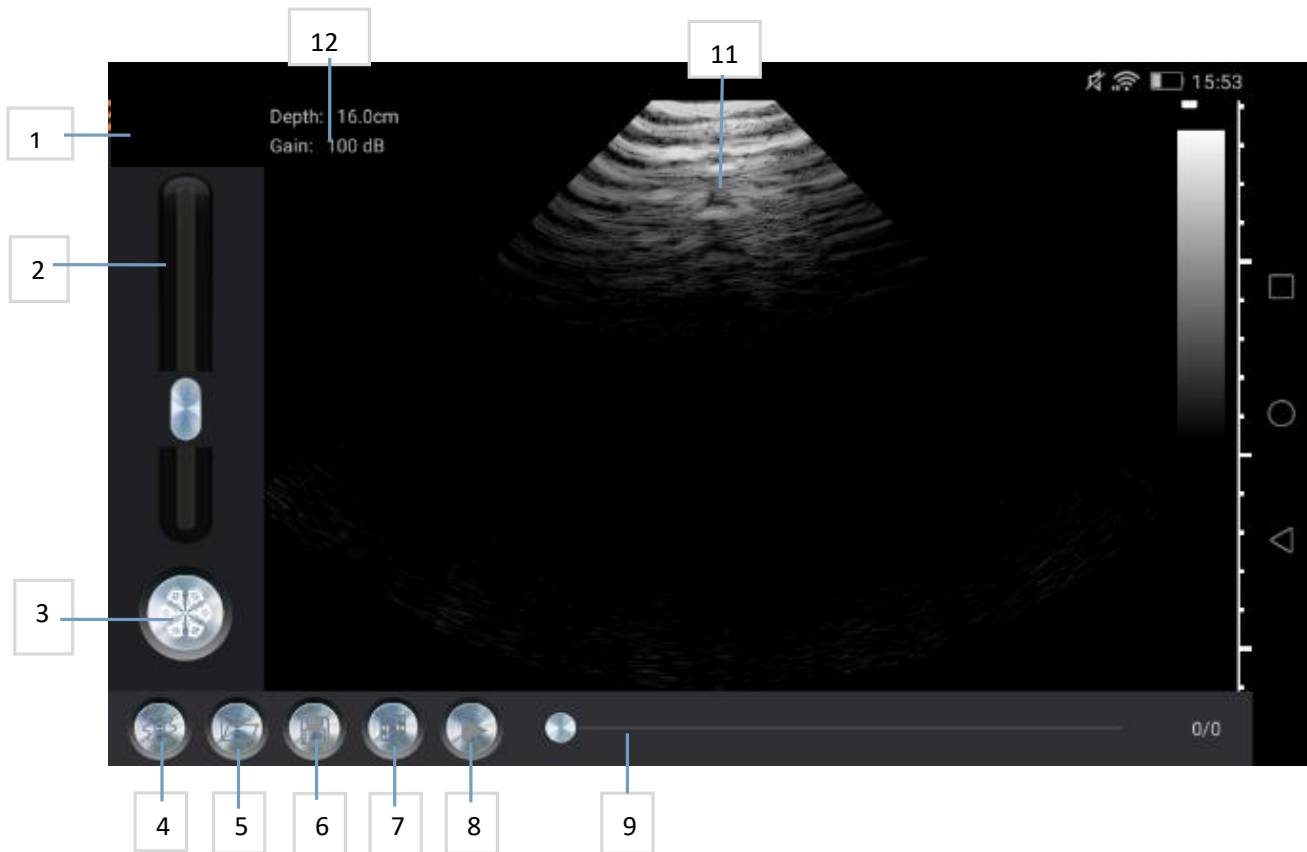
Image 3.1 Main Menu

1. Company Logo
2. Gain scale bar
3. Freeze button
4. Setting
5. File Exportation
6. Image saving
7. Video saving
8. Pause button
9. Video indicator
10. Colour
11. Image area
12. Depth and Gain information

After connecting the probe to the monitor, start the APP showing the main menu (there is no image when the APP is started for the first time).

Press working/freezing button to make the probe work or freeze, which is the same function as the button on probe. When in working status, slide up and down to adjust the image gain, to adjust the depth of scanning and to control the image area.

When in freezing status, the screen displays the number of images that are being played and the current image number. The user can use the slider to select the image. The four buttons on the right of slider are: setting, open file, save and play.

3.2 Image saving

When image displays in the image area, press the "save" button to save the image in the tablet.

Press the "browse image" button to select the saved image. If the user preferred to use the probe with the right hand (left hand), while operating the APP with left hand (right hand), adjust the interface in App.

3.3 Video saving

When there is an image displayed in the image area and the image is frozen, you can click the "Save Video" button to save the image to the display album, and click the "Browse Video" button to select the saved video. If you are used to holding the probe with your right hand (or left hand) and want to use your left hand (or right hand) to operate the APP, the APP provides Settings for adjusting the interface.

3.4 Backfat test procedures

1) To measure backfat, the gain is generally slightly lower (the near-field gain is relatively strong), measured, frozen, and then lengthened according to the display area of the near-field image, and the backfat thickness can be automatically obtained.

2) Backfat is the boundary between the fat layer and the muscle layer, so it is usually measured as the thickness of back fat. There are usually one, two or three layers of back fat at most. Of course, it varies from place to place.

3.5 Eartag number reading

After the Magascan is successfully connected to the tablet software and the machine is frozen, press the "Eartag Number Read" button on the machine to read, and align the sensor area of the probe to the ear tag.

If the correct ear signal is not read within 15 seconds, the device will make three short "beep" and exit the ear signal reading. You can press the "Eartag Read" button to restart reading.

If the correct ear signal is read, the device will beep "di...". , exit the reading of ear number, and display the read ear number on the main interface.

4. Maintenance:

4.1 Probe charger

This system supports two charging modes, USB charging and wireless charging.

Charge the probe when the battery is low. Remove the probe plug at the end of probe, use a USB cable to connect the probe and the USB charger (USB on PC or power bank). When charging, the battery indicator is red, when turning to be blue, battery charging is finished. Unplug the USB charging cable and insert the probe plug to keep the probe waterproof.

Wireless charging board is an optional accessory. Users can purchase a wireless charging board to charge the probe. The charging position is in the middle of the back of the probe. When charging, the probe shuts down automatically, and the power display sign flashes to indicate that it is charging.

4.2 Waterproof of monitor

The distributor/supplier could offer a monitor waterproof bag to protect the screen in dirty working conditions.

4.3 Clean

After each scanning, clean the top of probe with running water or wrap with damp cloth or tissue. The probe IPX-5, can be cleaned using running water for a while. DONOT immerse it in water.

4.4 Storage

When the probe is not in use, please keep the probe in a suitable package to avoid severe impact that may cause damage to the probe, and avoid the probe contacting too high temperature. The appropriate storage temperature is 0-40 degrees.

5. Specifications:

This chapter contains system and accessory specifications and standards. The specifications for recommended peripherals are in the manufacturers' instructions.

Product Description:

System weight: 318g/0.70lbs (including battery)
Dimensions: 19.3x4.7x3.5cm / 7.6x1.85x1.38 inch
Imaging Tech: Full digital beam forming
Display Mode: B
Gray Shades: 256 levels
Scanning Method: Mechanical Sector Scanning
Gain Control: 90-120dB, Multi levels adjustable
Depth Control: 100-180 adjustable
Backfat Thickness: Auto measurement, auto saving measurement results
Pre-Processing: Edge enhancement, frame rate related
Image Saving: Jpg format
Video Saving: Mp4 format
Pregnancy Report: PDF format
Testing Data: Excel format, pregnancy & backfat conclusion saved
Cine Loop: Max. 128 frames
Ear-number input : By manual; By Scanning barcorn ; By Scanning QR ; By reading RFID ear-tag
Built-in Battery: Rechargeable lithium battery
Probe Charger: AC240~110V, 5.0V, 2A

Environmental limits

Operating

System, battery, and transducer 10–40°C (50–104°F), 15–95% R.H.
700 to 1060hPa(0.7 to 1.05 ATM)

Shipping and storage

System and transducer
-35–65°C (-31–149°F), 15–95% R.H.
500 to 1060hPa (0.5 to 1.05 ATM)

Electromechanical safety

EN 60601-1:2003, European Norm, Medical Electrical Equipment–Part 1. General Requirements for Safety.

EN 60601-1-1:2001, European Norm, Medical Electrical Equipment–Part 1. General Requirements for Safety–Section 1-1. Collateral Standard. Safety Requirements for Medical Electrical Systems.

EN 60601-2-37:2008, European Norm, Particular requirements for the safety of ultrasonic medical diagnostic and monitoring equipment.

CAN/CSA C22.2, No. 601.1-M90: 2002, Canadian Standards Association, Medical Electrical Equipment–Part 1. General Requirements for Safety (including CSA 601.1 Supplement 1:1994 and CSA 601.1 1B-90:2002).

CEI/IEC 61157:2007, International Electrotechnical Commission, Requirements for the Declaration of the Acoustic Output of Medical Diagnostic Ultrasonic Equipment.

UL 60601-1 (1st Edition), Underwriters Laboratories, Medical Electrical Equipment-Part 1: General Requirements for Safety.

6. Quick Guide:

1. Take the scanner and pad out of the box.


2. Long press power button for 3 second (the light is flashing).

4. Open the scanner APP on the pad.

3. The scanner will be connected to APP automatically in 5 seconds.

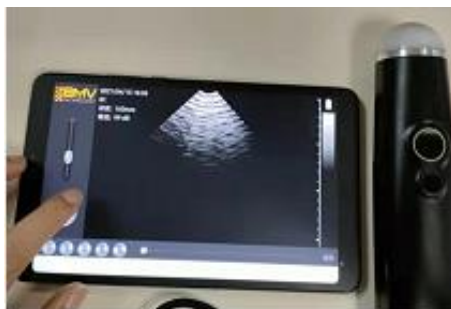5. Press "Freeze Button" to start scanning.

6. Slide up and down in the image area to adjust the depth.