



GENERAL INFORMATION

Lubri Gel is a water-based gel which on its medical procedures aspect is recommended during vaginal and rectal examinations. Aqua Lubri Gel is generally clear and odorless with a consistency slightly thicker than water to provide maximum comfort. In the medical field, it is commonly used during vaginal exams and trans-vaginal ultrasounds. It is added directly to the speculum or ultrasound device in order to make penetration more comfortable and to allow freer movement while examining the area. Using Aqua Lubri Gel may also avoid irritation from the equipment rubbing against the walls of the vagina. Rectal examinations may also require to ensure comfort and to prevent rectal tearing.

It does not contain formaldehyde and dyes, hypoallergenic, water-soluble, odorless, easily removable, doesn't stain, doesn't grease. It does not contain any active principles that might damage the probe transducer. The used preservatives are nontoxic and strictly in accordance with the law.

INSTRUCTIONS FOR USE

- * Gel is ready to use
- * Apply the desired amount of Aqua Lubri Gel
- * Connect and use the equipment and the probes by following the instructions supplied by the manufacturers.

- * After the procedure remove all residues of the gel with paper, TNT (Non-woven) or gauze moistened with water.
- * Stop using the gel for hypersensitive patients, if skin reactions occur.* Stop using the gel for hypersensitive patients, if skin reactions occur.

WARNINGS

- * Do not apply gel on skin affected by abrasions, erythema or wounds.
- * Close the bottle after use.
- * May cause irritations or reddenings with allergic persons.
- * Contact with eyes may cause irritation.
- * Analyst with sensitive skin should wear protective gloves during the use.
- * Do not use the gel after the "use by" date on the bottle.

INGREDIENTS: aqua, carbomer, sodium hydroxide, EDTA, methylisothiazolinone
methylchloroisothiazolinone, benzyl alcohol

STORAGE

Store at a dark place in room temperature. Keep container away from direct sunlight and sources of UV radiation.

EXPIRY

36 months

GUARANTEE – NOTICE

Magapor SL guarantees that the product complies with Directive 93/42/EEC and has been manufactured according to the procedures of Magapor SL Quality System certified ISO9001:2008 and EN ISO 13485:2012.